## Makrite® 910-N95







This respirator is only approved in the following configuration:

TC	Protection <sup>1</sup>	Respirator	Cautions and Limitations <sup>2</sup>
		910-N95	
84A-3323	N95	X	ABCJMNOP

Protection

N95 - Particulate Filter (95%) filter efficiency level) effective against particulate aerosols free of oil; Time use restrictions may apply.

- 2. Cautions and Limitations
- A Not for use in atmospheres containing less than 19.5% oxygen.
- B Not for use in atmospheres immediately dangerous to life or health.
- C Do Not exceed maximum use concentrations established by regulatory standards
- J Failure to properly use and maintain this product could result in injury or death.

  M All approved respirators shall be selected, fitted, used, and maintained in accordance
- with MSHA, OSHA and other applicable regulations
- N Never substitute, modify, add, or omit parts. Use only exect replacement parts in the configuration as specified by the manufacturer.
- O Refer to users instructions, and/or mainterance manuals for information on use and
- maintenance of these respirators.

  P NIOSH does not evaluate respirators for use as surgical masks.

## Particulate Respirator




#### **SERVICE TIME LIMITATION**

- Do not use the same respirator for more than 8 hours, continuous or intermittent, in dirty workplaces that could result in high filter loading.
- Leave contaminated area and remove respirator if breathing becomes difficult or proper fit cannot be obtained or the respirator is damaged, soiled, or distorted. Do not use the same respirator again once removed
- The maximum service time of this respirator is five (5) consecutive calendar days (including days of non-use), beginning from the first day of use.

Makrita Industries, Inc. • Hsi-chih, Taipei Hsien, Taiwan Phone No. in USA: 1-800-379-9929

# Makrite® 910-N95



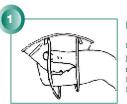

#### FITTING INSTRUCTIONS:

#### Step 1

Hold respirator in your hand as shown, with the nosepiece at your fingertips and the headbands hanging freely below your hand.



#### Step 2

Place respirator under your chin with the nosepiece up. Pull the top headband to top back of head.



Step 3

Pull the bottom headband overhead and place around neck below ears.



#### Step 4

Adjust headbands for comfortable fit Push the nosepiece inward with your fingertips to conform to the shape of your nose.

#### LIMITED WARRANTY

### IMPORTANT NOTICE TO PURCHASER:

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face heroof. The physical standards and specifications of Makrite will be met by products sold.

Exclusive Remedies: damages for the breach of this limited warranty are limited to the replacement of such quantity of Makrite, products proved to be defectively manufactured. Except as provided above, Makrite shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to we requestly the year.

#### **WARNINGS:**

- 1. Wearer must read fitting instructions and use limitations before use.
- This respirator does not protect against the risk of contracting disease or infection.
- Individuals highly sensitive to natural rubber latex may have an allergic reaction.
- 4. Before use, the wearer must be trained in the proper use in accordance with applicable safety and health standards for the contaminant and exposure level in the work area.

#### Caution:

The respirator should be fit checked for each wearing. Place both hands completely over the respirator and exhale. If air leaks around nose or the edges, adjust the nosepiece and/or headbands until a goo fit is achieved.

Makrita Industries, Inc. • Hsi-chih, Taipei Hsien, Taiwar

Makriton Industries, Inc. • Hsi-chih, Taipei Hsien, Taiwan